

TLM TaiLor Made™ Vertical Shock Foot

PDAC Approved (L5987)



Vertical shock optimized for your patients

The TaiLor Made (TLM) foot is designed with toe and heel elements that move independently along with a vertical shock mechanical spring pack. The TLM Control Hub provides superior vertical shock that can be adjusted to suit patients' needs, and it is lightweight to help users remain nimble enough for K3/K4 activities.

Features and benefits

- Independent movement between the heel and forefoot to create a smooth rollover and simulate plantarflexion/dorsiflexion, especially on inclines and declines.
- Shock absorption and comfort at heel strike from compression of the springs.
- Energy return at toe-off from release of the springs and the carbon fiber forefoot.
- Exchange the heel and forefoot springs for different stiffness levels to customize performance for your patient.
- Provides great terrain conformance to help patients navigate uneven surfaces more safely.
- THE lightest vertical shock foot available.



TaiLor Made™


3160 Southgate Commerce Blvd
Suite 38
Orlando, FL 32806
Ph (407) 930-5783
info@tlmprosthetics.com

tlmprosthetics.com


TLM Tailor Made™ Vertical Shock Foot

Foot customization

The vertical shock springs within the Control Hub and the carbon foot elements are configured and selected based on the patient's weight and activity level. But these elements can also be customized and exchanged by the CPO. A Boa® quick connection system and unique Control Hub hold all the elements in place and allow for easy assembly. Four different spring packs offer a variety of options and multiple toe and heel stiffness levels are available.

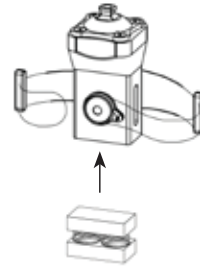

Control Hub with Boa® connection system loosened showing separate mechanical spring pack.

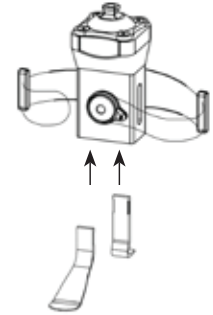

Independent heel and toe elements insert into the Control Hub after the spring pack.

Technical data

Size (s)	22 cm to 30 cm
Build height	7 in (18cm) with footshell
Heel height	3/8 in
Foot weight	16 oz without footshell
Weight limit	220 lbs (Normal/High Activity) 221 to 275 lbs (Normal Activity Only)
Footshell Color	Light/Brown

Individual Parts

TLM-FF-*	TLM Forefoot
TLM-HL-*	TLM Heel
TLM-SP-*	TLM Spring Pack
TLM-FS-*	TLM Footshell (Side/Size/(BR only if Brown Color)
TLM-BA	TLM Boa Connector
TLM-SL	TLM Elastic Sleeve
TLM-TOOL	TLM T6 Tool

Order example

Part number	Side	Size	Color
TLM	L	26-222	Blank for Light "BR" for Brown

Scope of Delivery

- TLM Foot
- BOA T6 Tool
- Footshell
- Individual Springs (2)
- Pyramid
- IFU
- Protective sleeve

Weight (lbs)	Activity Level	22	23	24	25	26	27	28	29	30
88-103	All	22-111	23-111	24-111						
104-121	Normal	22-111	23-111	24-111	25-121	26-121	27-121			
104-121	High	22-111	23-111	24-111	25-122	26-122	27-122			
122-143	Normal	22-111	23-111	24-111	25-121	26-121	27-121	28-231	29-231	30-232
122-143	High	22-112	23-112	24-112	25-122	26-122	27-122	28-232	29-232	30-232
144-165	Normal	22-112	23-111	24-111	25-122	26-122	27-122	28-232	29-232	30-232
144-165	High	22-112	23-112	24-112	25-222	26-222	27-222	28-232	29-232	30-232
166-192	Normal	22-112	23-112	24-112	25-122	26-122	27-122	28-232	29-232	30-232
166-192	High	22-113	23-113	24-113	25-223	26-223	27-223	28-233	29-233	30-233
193-220	Normal				25-223	26-223	27-223	28-233	29-233	30-233
193-220	High				25-223	26-223	27-223	28-234	29-234	30-234
221-249	Normal				25-323	26-323	27-323	28-333	29-333	30-333
221-249	High									
250-275	Normal				25-324	26-324	27-324	28-334	29-334	30-334
250-275	High									

Part Formula: Size of foot (cm) - (Category Forefoot) (Category Heel) (Category Springs)
Activity Level: Normal, High

Weight (lbs)	Activity Level	22	23	24	25	26	27	28	29	30
88-103 All	All	22-111	23-111	24-111						
104-121	Normal	22-111	23-111	24-111	25-121	26-121	27-121			
104-121	High	22-111	23-111	24-111	25-122	26-122	27-122			
122-143	Normal	22-111	23-111	24-111	25-221	26-221	27-221	28-231	29-231	30-232
122-143	High	22-112	23-112	24-112	25-222	26-222	27-222	28-232	29-232	30-232
144-165	Normal	22-112	23-211	24-211	25-222	26-222	27-222	28-232	29-232	30-232
144-165	High	22-212	23-212	24-212	25-322	26-322	27-322	28-332	29-332	30-332
166-192	Normal	22-212	23-212	24-212	25-222	26-222	27-222	28-232	29-232	30-232
166-192	High	22-213	23-213	24-213	25-323	26-323	27-323	28-333	29-333	30-333
193-220	Normal				25-323	26-323	27-323	28-333	29-333	30-333
193-220	High				25-323	26-323	27-323	28-334	29-334	30-334
221-249	Normal				25-323	26-323	27-323	28-333	29-333	30-333
221-249	High									
250-275	Normal				25-324	26-324	27-324	28-334	29-334	30-334
250-275	High									

Part Formula: Size of foot (cm) - (Category Forefoot) (Category Heel) (Category Springs)
Activity Level: Normal, High